ALLURE CARBON

ALLURE CARBON FIT

Manual

FR	2 - 18
EN	19 - 35
IT	36 - 52
DE	53 - 69
ES	70 - 86

FR

Sommaire

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EN

Table of contents

- 1- Safety tips.**
- 2- Various models in the range.**
- 3- Various accessories.**
- 4- How to install tires on the wheels?**
- 5- How to install wheels on the chair?**
- 6- How to install compatible handrails on Fit wheels?**
- 7- Inflation pressure.**
- 8- Wheel maintenance.**
- 9- Transportation and storage.**
- 10- Lifespan.**
- 11- Important information.**
- 12- Warranty.**
- 13- Contact.**

Dear Customer,

Congratulations on choosing to equip your wheelchair with a CORIMA product. We invite you to carefully read this manual, which contains important information regarding the assembly, use, and maintenance of your wheels or accessories. Proper use and regular maintenance of your wheels and accessories will enhance the lifespan of the products.

IMPORTANT : Before using your CORIMA product, read and carefully follow the instructions for use.

This CORIMA product has been designed and manufactured exclusively for mounting on active wheelchairs intended for use on smooth, well-maintained paved roads or in everyday situations with uniform surfaces.

Any other use of this product, such as off-road, gravel, sports, or on rough terrain, could damage it, lead to malfunctions, or result in a serious, even fatal accident.

CORIMA pneumatic wheels are not compatible with tubeless conversion kits. Only tires with inner tubes are accepted (except for latex inner tubes).

On our active wheelchair wheels, the serial number is located on the rim and inside the rim well.

In case of a fall or a severe impact, it is imperative to have your wheel checked by our technicians.

1- Safety tips.

1.1 BEFORE USING YOUR WHEELCHAIR

- Check the pressure and condition of your tires, ensuring that the inflation pressure complies with the type of tire(s) and wheel(s) you are using. (see Chapter 4 and 7)
- Verify that the levers are correctly adjusted and engaged. (See Chapter 5/)
- Check that your wheel rim is not damaged.
- For the version compatible with commercially available handrails, ensure that the handrail is correctly installed. (See Chapter 6)

1.2 GENERAL RULES

- Do not remove the stickers placed on the rim related to usage and safety instructions.
- Follow Periodic Maintenance advice
- Use only genuine CORIMA replacement parts
- Do not perform any interventions or modifications on your wheel such as drilling, sanding, etc...
- Any damaged parts due to impacts or accidents should be replaced with CORIMA spare parts and not repaired.
- In rainy weather or on wet surfaces, braking distances are extended. Consider this and plan ahead.

- In case of a flat tire, avoid riding with a flat tire to prevent damage to the carbon rim.

- For CORIMA wheels, the combined weight of the user/ wheelchair must not exceed 130 kg.»

2- Various models in the range.


2.1 ALLURE CARBON

ALLURE CARBON DAILY 4 SPOKE WHEELCHAIR **R2329252008**
WHEEL 24" CLINCHER 1/2" AXLE LEFT + PUSHRIM

ALLURE CARBON DAILY 4 SPOKE WHEELCHAIR **R2329251008**
WHEEL 24" CLINCHER 1/2" AXLE RIGHT +
PUSHRIM

2.1.1 DIMENSIONS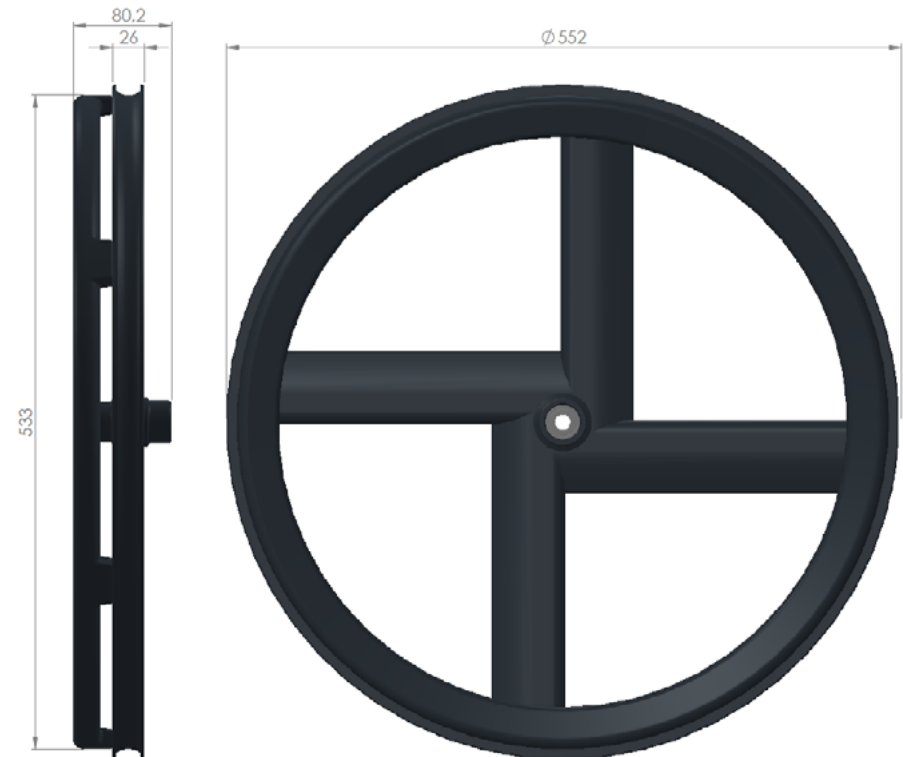

The axles (quick releases) and tires are not provided.

2.2 ALLURE CARBON FIT


ALLURE CARBON FIT DAILY 4 SPOKE WHEELCHAIR
WHEEL 24" CLINCHER 1/2" AXLE LEFT **R2329457015**

ALLURE CARBON FIT DAILY 4 SPOKE WHEELCHAIR
WHEEL 24" CLINCHER 1/2" AXLE RIGHT **R2329456015**

The axles (quick releases), tires, and handrails are not provided.

2.2.1 DIMENSIONS

The dimensions below are provided as an indication for the user.

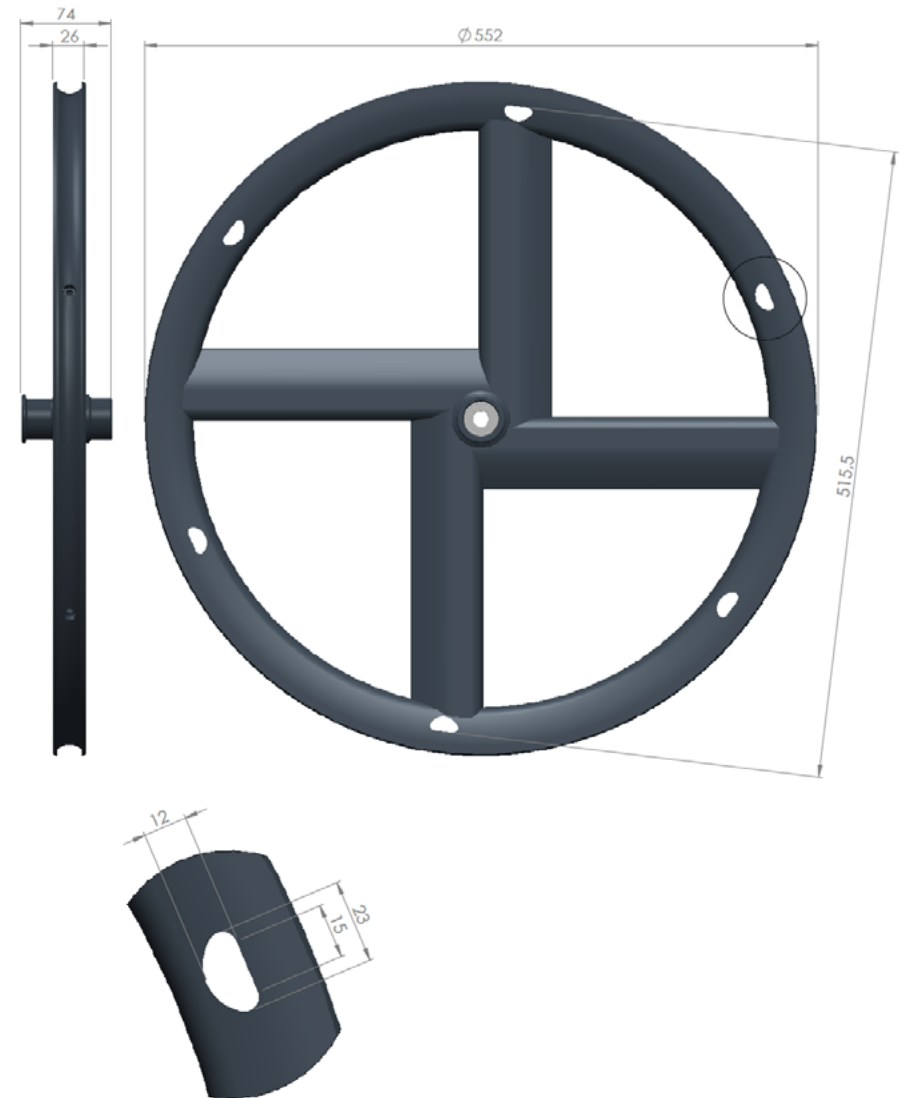3- Various accessories.

3.1 COMMON TO ALL PRODUCTS

00029034

QR AXLE D1/2" X 99 MM, STAINLESS STEEL, BLACK ALU RELEASE BUTTON, «CORIMA» LOGO FOR DAILY WHEELCHAIR

00029035

QR AXLE D1/2" X 104 MM, STAINLESS STEEL, BLACK ALU RELEASE BUTTON, «CORIMA» LOGO FOR DAILY WHEELCHAIR

00029036

QR AXLE D1/2" X 99 MM, TITANIUM, BLACK ALU RELEASE BUTTON, «CORIMA» LOGO FOR DAILY WHEELCHAIR

00029037

QR AXLE D1/2" X 104 MM, TITANIUM, BLACK ALU RELEASE BUTTON, «CORIMA» LOGO FOR DAILY WHEELCHAIR

RK00028506

SCHRADER VALVE EXTENSION 8.5MM DIAMETER 20.8MM LENGTH 24" AND 25" WHEELS

RK270231

KIT OF 2 BEARINGS R8 2RS FOR 1 WHEEL (DAILY 1/2")

3.2 SPECIFIC TO THE ALLURE CARBON FIT WHEELS

RK00028261

KIT 6 SCREW COVER FOR 1 WHEEL "FIT"

RK00028526

COMPLETE KIT FOR 1 WHEEL "FIT"

4- How to install tires on the wheels ?

To guide you through the various steps, a tutorial is available online through the following link.

[Tutorial Link](#)

YOUTUBE

@corima_wheels :

Allure Carbon Mounting

The ALLURE CARBON and ALLURE CARBON FIT wheels have been designed with a leading edge and a trailing edge to facilitate daily movements. Thus, we distinguish between right wheels and left wheels, each having a preferred direction of rotation.

Example of a right wheel.

Narrower rear part: trailing edge.

Wider front part: leading edge.

These are stamped with an R (right) and L (left) label to distinguish them.

5- How to install the wheels on the wheelchair?

To guide you through the various steps, a tutorial is available online through the link below.

[Tutorial Link](#)

YOUTUBE

@corima_wheels :


Allure Carbon Mounting

Diagram of axle + hub dimensions

A : Quick release axle length (also known as skewer).


B : Wheelchair hub length.

The length **A** corresponds to length **B** + 53.8 mm.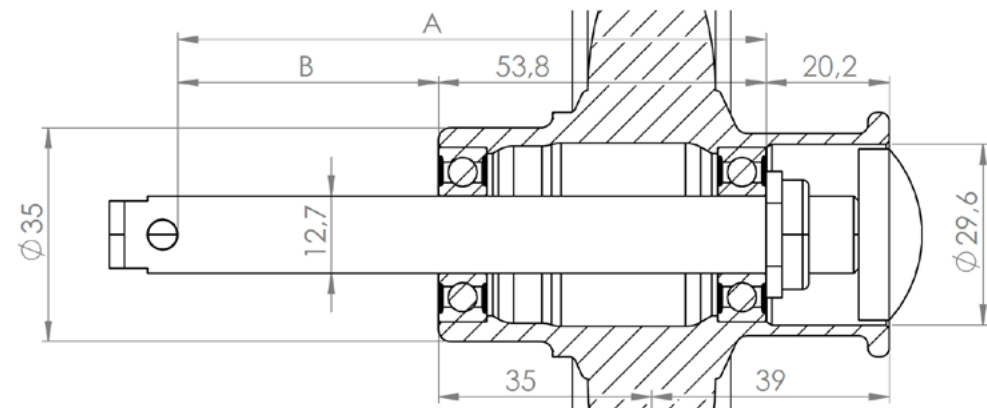

6- How to install a compatible handrail on Allure Carbon Fit wheels ?

To guide you through the various steps, a tutorial is available online through the link below:

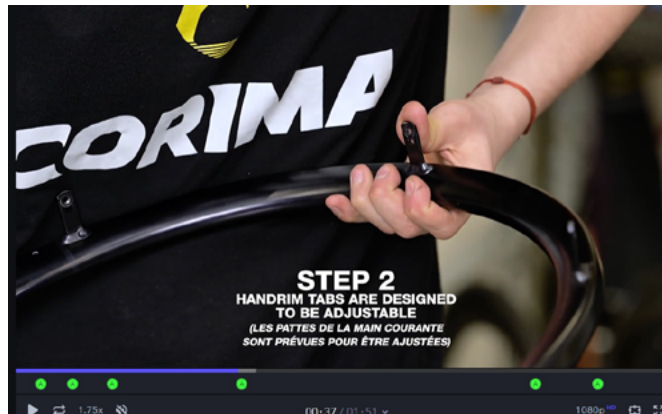[Tutorial Link](#)

YOUTUBE

@corima_wheels :

Allure Carbon Fit Handrim Mounting

Note that ALLURE CARBON FIT wheels are compatible with handrails equipped with the standard 6 tab mounts, which is the most common (mounting bracket system). The suitable diameter for the 6 tab mounts is 512mm. The maximum width of the mounting brackets should be 15mm with holes centered on the bracket. The assembly is suitable for M5 screws. The maximum thickness of the mounting brackets is 3mm. The wheels have been developed for use with a maximum camber of 4 degrees.

Example of a Carbolife handrail with the standard 6 tab mounts.

7- Inflation pressure.

Depending on the tire width indicated on the tire, refer to the table below. The minimum tire section is 25 mm, and the maximum is 37mm. For comfort and ease, we recommend fitting a 25mm or 28mm tire for our 26mm wide rims. The CORIMA rim tape is provided by default and is mandatory. If it is damaged, it should be replaced. CORIMA rims are not compatible with tubeless conversion kits. Only tires with inner tubes are accepted (excluding latex inner tubes). Inflation using CO2 cartridges is permitted only if the flow is adjustable. Instant inflation cartridges are prohibited with CORIMA wheels.

Size	Max pressure
25-540 / 24 x 1.00	9 bars / 130.5 PSI
28-540 / 24 x 1 1/10	8 bars / 116 PSI
30-540 / 24 x 1.20	7 bars / 101.5 PSI
32-540 / 24 x 1 3/8 x 1 1/4	6 bars / 87 PSI
37-540 / 24 x 1 3/8	5 bars / 72.5 PSI

Caution!: If the tire manufacturer specifies an inflation pressure lower than that in the table, the lower value should be used.

8- Maintenance of the wheels.

Any impact can cause visible or invisible damage, often repairable by our composite after-sales service, but we cannot be held financially responsible. In cases where a substantial impact has occurred, it is imperative to have your wheel checked by our technicians.

Regularly check the wear and tear of your wheels and rims. Do not use your wheels if you notice cracks on the rim and/or hub, if the rim is chipped, or if it is deformed.

Regarding rim wear, an annual check of the wheels is recommended. CORIMA will assess the degree of wear and can indicate the level of reliability of the rims. Before each use, check the tire pressure.

For wheel cleaning, do not use a high-pressure water device, especially for hub cleaning, where pressurized water could penetrate and degrade the bearings or components of your wheel. Clear or slightly soapy water is suitable; do not use products that could chemically degrade the epoxy resin of the composite, the coating, or the stickers.

To dry your wheel, use a cloth. Do not use acetone.

Remember to grease your hub axle. Use grease resistant to oxidation and aging, ensuring good protection against corrosion, for example, a conventional cycle grease. Do not use oil that would dilute the grease in the bearings. Also, grease the outer clamping nuts.

The timelines and distances are indicative and may change significantly depending on usage conditions and the intensity of your movements. Find the list of spare parts for your wheel on our website. Changing the bearings of your wheel should be done by a retailer or by CORIMA.

Failure to follow maintenance and usage instructions could damage your wheel or lead to malfunction. Inspection and maintenance are important for your safety and the lifespan of your wheels and wheelchair.

A poorly maintained wheelchair and its components can break or malfunction, potentially causing an accident where you could be killed and/or seriously injured.

9- Transport and storage.

Ideally, transport your wheels in protective covers designed for this purpose to shield them from impacts. Avoid exposing your wheels to high temperatures (in direct sunlight, behind car windows, too close to an exhaust outlet, etc.). Never suspend your CORIMA wheels.

10- Lifespan.

The lifespan of CORIMA wheels depends on the usage of each of our customers and the specific conditions they apply. Under normal usage and maintenance conditions, in the absence of impacts or excessive stress, our wheels retain their mechanical characteristics for several years beyond the contractual warranty period. However, we recommend our customers to replace their wheels with new ones every 5 years. Some components such as bearings, valves, and rim tape are considered consumables for which the contractual warranty does not apply. Regarding wheel wear, an annual check of the wheels is recommended by your authorized CORIMA retailer or directly at CORIMA. They will assess the degree of wear and indicate the reliability level of the rims. The frequency of this check depends on the product's usage, and we are available to advise if you use your wheels intensively. If any part of your wheel shows signs of fatigue, you should replace it immediately.

AFTER-SALES SERVICE

Despite the utmost care in our manufacturing, if a defect appears or if a repair is necessary, contact either your authorized CORIMA retailer, bringing the defective product along with your purchase invoice and specifying the malfunction. Alternatively, contact our customer support at support@corima.com.

11- Important information.

Caution ! :

- CORIMA products are designed and optimized for users whose weight and that of their equipment do not exceed 130 kg.
- Before moving, familiarize yourself with the operation of your new wheels.
- After an impact or collision, we strongly recommend, for your safety and the safety of others, that you have your wheels inspected.

To reduce the risk of serious injury, ensure that you maintain your wheels and wheelchair according to the instructions, install and adjust all components correctly. Be aware of the dangers of the road and traffic. Familiarize yourself with the laws in force in the country where you are. If you ride at night, it's not just about seeing but being seen. Reflectors are not sufficient and do not replace lighting. Wearing reflective clothing is strongly recommended, even mandatory in some countries.

12- Warranty.

12.1 LEGAL WARRANTY

CORIMA, through its authorized agents and distributors in the country where the product was purchased, warrants its products against non-conformities and hidden defects (1) for a period of two years (2) from the date of purchase. (1) Hidden defects only concern French legislation. §§ 1641 to 1649 of the Civil Code. (2) Some countries or states may allow a longer duration for an implied warranty, and/or for the exclusion or limitation of direct or consequential damages, which means that the limit described above would not apply. This limited warranty gives you specific legal rights, and you may also have other rights that vary by local laws.

12.2 COVERAGE

The warranty applies only if the product is assembled in accordance with the instructions provided with the product, maintained according to our recommendations, and used correctly. This limited warranty does not apply to normal wear and tear*, to parts that have been refurbished or subjected to inappropriate repairs or modifications, nor to damage caused by an accident, misuse, or lack of reasonable care. Among the misuses that render the warranty void are jumping, acrobatics, sports, and similar activities.

The sole remedy under the CORIMA limited warranty or any implied warranty is limited to the repair or replacement, at CORIMA's sole discretion, of all or part of the affected product. Under no circumstances can CORIMA and its agents or distributors be held liable for direct or consequential damages, or for covering the transportation or shipping costs of the product, whether the claim is based on contract, warranty, negligence, or product liability, including, without limitation, losses or damages related to property other than the warranted product itself, loss of use of property, or any other financial loss. Regardless of the cause, neither CORIMA nor any agent or distributor may be subject to any financial contribution or compensation for a product. Rights granted by the warranty and benefits cannot be assigned or transferred. CORIMA's obligations under any warranty are limited, to the fullest extent permitted by law, to those specified in this CORIMA limited warranty.

The warranty is void if the product:

- Underwent a technical and/or structural modification, if it was repaired or repainted outside an authorized CORIMA repair center.
- Was repaired and/or used with unauthorized spare parts.
- The product's serial number has been removed, erased, altered, or made illegible.
- Components other than CORIMA mounted on our wheels are subject to the same legal warranty conditions. Claims for the warranty of any components must be made to the respective manufacturers or distributors. Under no circumstances can CORIMA and its agents or distributors be held liable for direct or indirect damages caused by a product not designed, manufactured, and marketed by CORIMA.
- Components considered as normal wear and tear: hubs, tires, inner tubes, rim tape, bearings.

12.3 LIFETIME RIMS WARRANTY

The quality and reliability of our wheels, which we control from design to marketing, allow us, through our authorized agents and distributors, to offer you, as an extension of legal guarantees, a free lifetime warranty on the rims that make up the new wheels purchased as an accessory or on a wheelchair from June 1, 2023, and valid for the 2022 and subsequent ranges. The lifetime warranty is not offered on 'Factory Outlet' wheels sold on our E-shop or through our authorized CORIMA dealers. However, these wheels are covered by the two-year legal warranty.

12.4 CONDITIONS

To validate the lifetime warranty, the wheel must be registered on the CORIMA website, under the Support / Registration tab. Registration link: <https://www.corima.com/fr/support/garantie.html>

Wheel registration should not exceed 30 days from the date of purchase. The registration form must be correctly filled out. If you register your wheel within ten days of the purchase date, you will also receive:

- **Our 'Contrat Confiance' replacement program.**

12.5 COVERAGE

This lifetime warranty is only valid for the initial purchaser and is not transferable. Please note that the lifetime warranty does not cover decoration and finishing, or components other than the rim. Under the lifetime warranty, CORIMA is limited to the repair or replacement, at CORIMA's sole discretion, of all or part of the affected product.

12.6 CRASH REPLACEMENT

In the event of an impact, fall, or road accident unrelated to normal use, it can be challenging to discover broken wheels and be required to invest entirely in a new pair. For these reasons, CORIMA offers a crash replacement program for your CORIMA wheel purchased from June 1, 2023, onwards, valid for the 2022 and subsequent ranges. This program allows you to purchase a new wheel with a discount from the range of the year of the request.

12.7 CONDITIONS

To benefit from the crash replacement program, your wheel must be registered within 10 days of the purchase date. Submit your wheel along with the purchase invoice to your dealer, who will handle the request with our After-Sales Service, the only entity authorized to validate the crash replacement based on the analysis of the provided elements. The crash replacement request cannot exceed 3 years from the purchase date, and

the discount offer will be decreasing based on the purchase date. This program is reserved for the initial user and is not transferable. Obtaining the crash replacement requires the return of the wheel and cannot be retained by the customer. Failure to return the wheel will cancel this offer.

12.8 COVERAGE

The crash replacement program covers the entire rim. Dealer labor costs, wheel refurbishment, and transportation expenses are the responsibility of the user. A wheel obtained through the crash replacement program cannot be subject to a second request but is eligible for legal warranties and the lifetime warranty. The wheel obtained through the crash replacement program will be automatically registered by our services.

12.9 OFFER

The discounts from the offers indicated below will apply to the current recommended retail prices in your country at the time of the crash replacement program request.

50% within 12 months of the purchase date.

30% within 13 and 24 months of the purchase date.

10% within 25 and 36 months of the purchase date.

12.10 SPARE PARTS

Spare parts are available from our authorized CORIMA retailers during the product's marketing period, with availability for 7 years from the date of CORIMA's end of marketing for that product.

12.11 COUNTERFEIT

WARNING ABOUT COUNTERFEIT PRODUCTS

THE USE OF COUNTERFEIT PRODUCTS IS VERY DANGEROUS AND CAN CAUSE SERIOUS FALLS RESULTING IN SIGNIFICANT OR EVEN FATAL INJURIES TO YOU AND OTHERS.

CORIMA products are protected by industrial property rights. Therefore, in the case of the purchase, use, or sale of a counterfeit product, we reserve the right to take any measures we deem appropriate to cease the infringement of our rights and seek compensation for the damages incurred, as defined in Articles L613-3 to L613-6, L615-14, and L716-9. It goes without saying that counterfeit products do not benefit from any CORIMA warranties and result in the cancellation of all CORIMA warranties on a genuine CORIMA product used with counterfeit products. This also implies the cancellation of all accidental, material, moral, civil, and physical liabilities on the part of CORIMA. The vast majority of CORIMA products are manufactured in our factories outside of Asia. The sale of products on exotic sites is likely to involve counterfeit products.

To avoid any scams, and without these tips being exhaustive, we advise you to be very vigilant:

On websites offering CORIMA-branded products at prices significantly below the market.

In peer-to-peer sales that do not provide a purchase invoice, maintenance history, and information on the origin of the product.

Despite all the care taken in our manufacturing, if a defect appears or if a repair is necessary, contact your authorized CORIMA dealer, providing them with the defective product along with your purchase invoice and specifying the malfunction. You also have the option to contact CORIMA directly through our [support: https://support.corima.com/](https://support.corima.com/)

13 Contact.

For over 30 years, we have been developing concepts that ensure the reliability and accuracy of our wheels in all conditions. While this constant pursuit of innovation and performance is deeply embedded in our DNA, we also give supreme importance to a perhaps less tangible element, but one that has gracefully carried us since our very beginnings: TRUST.

The exceptional relationships we maintain with our customers, suppliers, and athletes are an integral part of the CORIMA identity: we are a family. Our brand can proudly rely on the strength of these lasting partnerships, allowing us to continue designing the best wheels on the market.

For any assistance, you can contact us through our customer support at <https://support.corima.com/>, providing the serial number and/or photos for verification.

This manual may not be used or reproduced, even partially, in any form or by any means, without the prior written permission of CORIMA. This material complies with the provisions of Directive 93/42/EEC.

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